


(1)報核仿單標籤以粘貼全型實物為原則。

註 (2)仿單標籤等實物過大或印於玻璃金屬容器等不便於粘貼
時得附送現品並將照相影本代替粘貼報核。

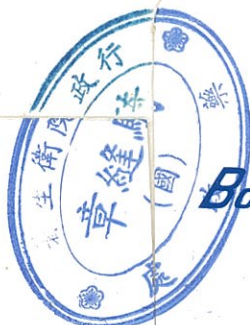

Boxley®

Boxley®

**'Tai Yu' Boxley
Cleaning Solution**

For use with
Soft (Hydrophilic)
Contact Lenses Only

45ml

Boxley®

Cleaning Solution

**FOR USE WITH
SOFT CONTACT LENSES**

Boxley®

Boxley®

DESCRIPTION:

"Tai Yu" Boxley Cleaning Solution is a sterile isotonic Solution for daily cleaning of soft (hydrophilic) contact lenses.

CONTENTS:

A sterile, buffered, isotonic aqueous solution containing nonionic surfactants with thimerosal 0.004% and edetate disodium 0.2% added as preservatives.

CAUTION:

Not intended for use in the eye. All sterile solutions must kept in their original containers to avoid contamination. Do not touch dropper tip of bottle to any surface since this may contaminate solution, keep cap closed when solution is not in use. Store at room temperature.

DIRECTIONS:

Read accompanying literature.

STORE AT ROOM TEMPERATURE

LOT
MFG
EXP

Boxley®

"台裕" 寶世利

保視力清潔液

衛署醫器製字第000368號

軟性隱形眼鏡專用

適合各廠牌軟性隱形眼鏡使用

45ml

保視力

清潔液

◎瑞隆高分子股份有限公司 註冊商標
台裕化學製藥股份有限公司 製造
TAI YU CHEMICAL & PHARMACEUTICAL CO., LTD.
新竹縣竹東鎮興山路十一巷三六號

軟性隱形眼鏡適用

Boxley®

用途:

本清潔液專供軟性隱形眼鏡使用。

說明:內含具有清潔及散化污垢功效之Poloxmer-407非離子性表面活性劑,及能使清潔作用發揮之增稠劑:PVA及Hydroxyethylcellulose。另加Thimerosal及Edetate disodium以抑止黴菌及細菌之生長。

注意:

- (1)加熱消毒法與藥水消毒法不可混合使用。
- (2)本劑不可點眼。
- (3)使用後,請蓋緊瓶蓋,在室溫下保存,不必冷藏。
- (4)請勿觸摸瓶口,或將本液轉入其他容器中,避免污染。
- (5)對汞類製劑過敏者不可使用。
- (6)運送及儲藏過程中,須防止包裝損壞。
- (7)露天存放須避免強光照射及污染。
- (8)應預防害蟲之侵襲。
- (9)須防止潮濕,高熱,有害輻射及化學腐蝕等之侵害。

使用方法:

請詳閱盒內說明書後使用。

*** 在室溫保存**