



“雷速”重碳酸鹽粉

NOT FOR PARENTERAL USE

This package contains: 1.34 Kg Sodium Chloride U.S.P.
3.74 Kg Sodium Bicarbonate U.S.P.

See SB-1000 series acid concentrate label for final dialysate concentrations when properly diluted with Purified Water (AAMI quality or equivalent) in a three stream 36.83X bicarbonate proportioning artificial kidney (hemodialysis) machine.

Mixing Instructions

1. Empty contents of one package into clean, disinfected mixing container.
2. Add Purified Water (AAMI quality or equivalent) to bring total volume to 15 U.S. gallons (56.8 liters) in size.
3. Mix well. Keep mixing until completely dissolved.
4. Analyze dialysate for correct concentrations and read SB-1000 series acid concentrate label prior to dialysis.

CAUTION: BC-1-15 bicarbonate concentrate can only be used in 36.83X bicarbonate proportioning machines with RENASOL® SB-1000 series acid concentrate or equivalent. Refer to the hemodialysis machine operator's manual for instructions prior to starting dialysis. Do not use if package is damaged. **Do not use with 45X dilution dialysate delivery system.** Bacterial growth may occur in concentrated bicarbonate solutions. Take care to avoid contamination. Disinfect all container, machines, transfer lines, etc., which contact the solution. **Use with 48 hours of preparation.** Store under 40°C in a sealed container after preparation. Federal (U.S.A.) law prohibits dispensing without prescription. Failure to follow the Instructions for Use may result in patient injury.

許可證字號: 衛署藥輸字第019534號
處方: 本藥限由醫師使用
藥商名稱: 佳醫健康事業股份有限公司
地址: 新北市中和區中正路880號17樓
電話: (02)2225-1888

LOT:

MFD:

EXP:



Manufactured in USA by:

Medivators Inc.
14605 28th Avenue North
Minneapolis, MN 55447 USA
Tel: +1.763.553.3300
Toll Free: +1.800.328.3340
Fax: +1.763.553.3387
email: info@medivators.com

67720-250/C