

87.11.1

號

VISCONE Peroxide Neutralizing Tablet "SINPHAR"

Actions :

They are used to neutralize the Peroxide solution after disinfection occurs.

Composition : Each Tablet Contains :

Catalase70U

Indications(Uses) :

Neutralization of hydrogen peroxide solution for the disinfection of contact lenses.

Directions :

Follow the instructions below each time you remove your contact lenses. Always wash, rinse and dry hands before handling contact lenses.

1.Clean, and Rinse your lenses :

Remove and handle one lens at a time. Gently rub each lens with a daily cleaner. Rinse thoroughly with Saline solution. Place the lens in the appropriate basket of the lens holder attached to the cap and close the basket lid.

2.Disinfect and Neutralize your lenses :

Remove one tablet into the lens case cup filled with Peroxide solution. Immediately place the lens holder and tighten the cap. Make sure that the tablet is in the solution. You will see a few bubbles or some light bubbling. If the tablet immediately begins to bubble vigorously, causing foam to form on the solution surface, the tablet is neutralizing the solution too quickly for disinfection to take place. Discard the solution and tablet, fill the cup with fresh solution, add another tablet and immediately tighten the cap. Allow the lenses to soak for a minimum of 4 hours to 16 hours. The delayed-release Viscaue Peroxide Neutralizing Tablet will allow disinfection to occur before it neutralizes the solution. During the neutralization process, the tablet will begin to bubble vigorously and will completely dissolve.

3.Wear :

After soaking, apply the lenses directly to your eyes. If desired, rinse your lenses with sterile saline solution before lens application.

Precautions :

- 1.Do not use Viscone Peroxide Neutralizing Tablet if you are allergic to the enzyme catalase.
- 2.Always neutralize lenses which are peroxide disinfected with Viscone Peroxide Neutralizing Tablet before applying lenses to your eyes.
- 3.Do not use tablets that appear to be broken or chipped.
- 4.Never use Viscone Peroxide Neutralizing Tablet in a heat disinfection unit as these products are not designed for use with heat disinfection.
- 5.Keep all tablets out of reach of children.
- 6.If you experience eye discomfort, excessive tearing, vision changes, or redness of the eye, immediately remove your lenses and promptly contact your eye care practitioner.
- 7.Store tablets at room temperature.

"SINPHAR"

VISCONE Peroxide

成份：每錠含：

Catalase

效能：隱形眼鏡

使用方法：請詳閱說明書

本品若有破損或完封

衛署醫器製字第 000

SINPHAR PHARMAC



SINPHAR PHARMACEUTICAL CO., LTD.
84, CHUNG SHAN VILLAGE TUNG - SHAN SHINE, I - LAN, TAIWAN

※外文仿單應檢附中文譯本

0011674