


## \*Tips for use\*

- \* The AQUACEL® dressing will shrink slightly as it absorbs wound fluid and gels.
- \* AQUACEL® dressing is nonadhesive and will need to be covered with a moisture-retentive dressing (such as DuoDERM® Extra Thin dressing, DuoDERM Signal® dressing, or Versiva® dressing) or another appropriate secondary dressing. You may be instructed to change the outer dressing if it becomes saturated between doctor visits.
- \* If you notice leakage escaping from your dressing, call your doctor or healthcare provider and they will change the dressing. In case you have to change the dressing yourself, wash your hands before and after the dressing change and follow the instructions provided by your doctor or healthcare provider.
- \* It is important that you do not pull your dressing away from your skin or peek under the dressing to check your wound, as this can let in harmful bacteria and may increase the chance of infection. While the dressing may have to be changed every 2 to 3 days in the beginning, it can be left on the wound for up to 7 days in the later healing phases.

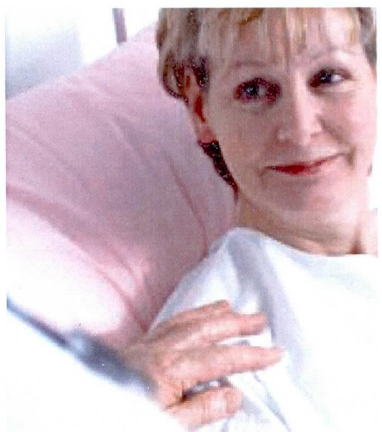

Call your doctor or healthcare provider immediately if you notice any of the following symptoms:

- Fever
- Tenderness or redness of skin around the wound
- Change in the color of drainage
- The wound or the surrounding skin feels hot or is swollen
- Unusual/excessive wound pain

\*Please see package insert for full product information, including instructions for use.