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

fm system-workstations

For intravenous anaesthesia and fluid management in intensive care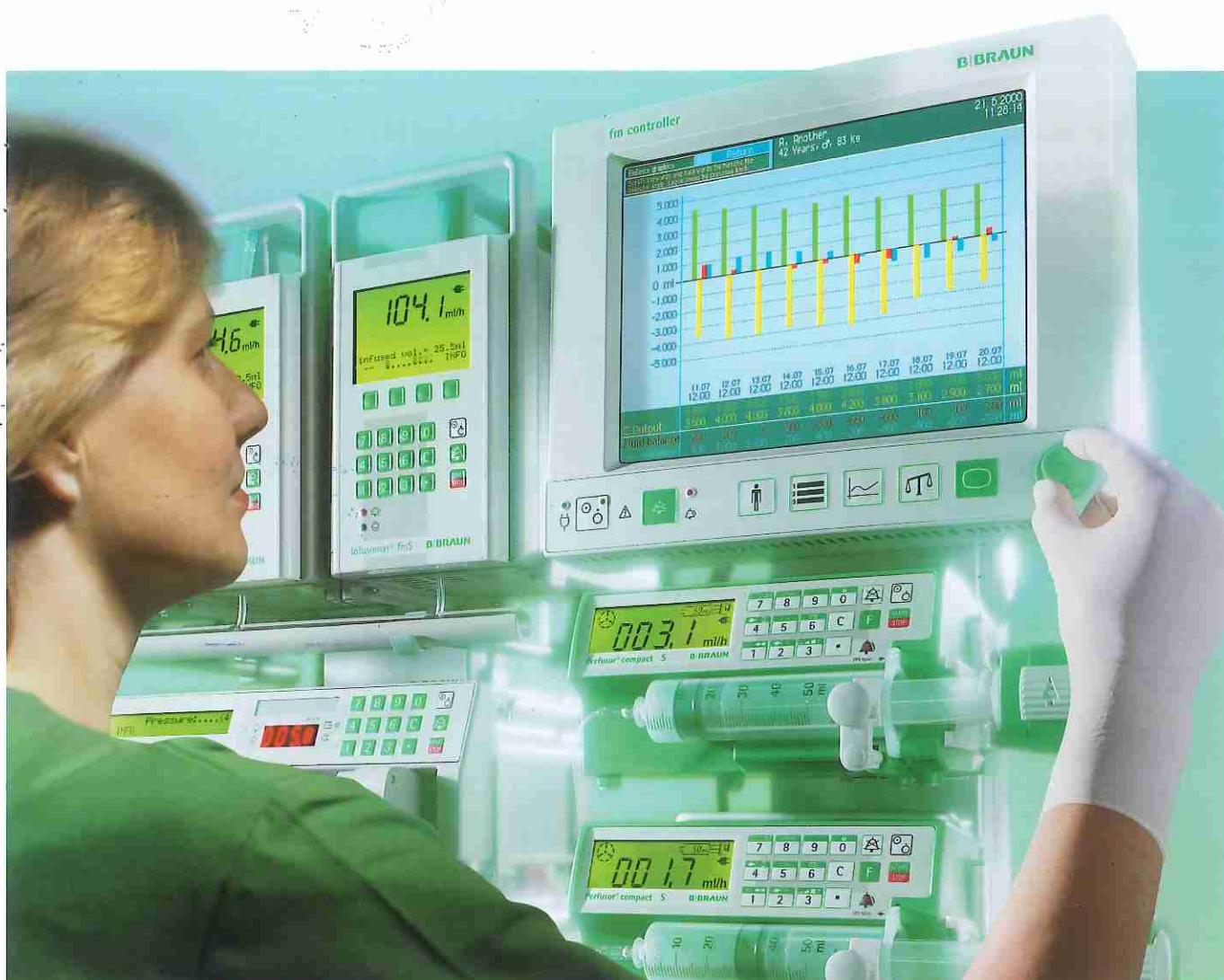


Leading in application, operation
and documentation

A modular concept...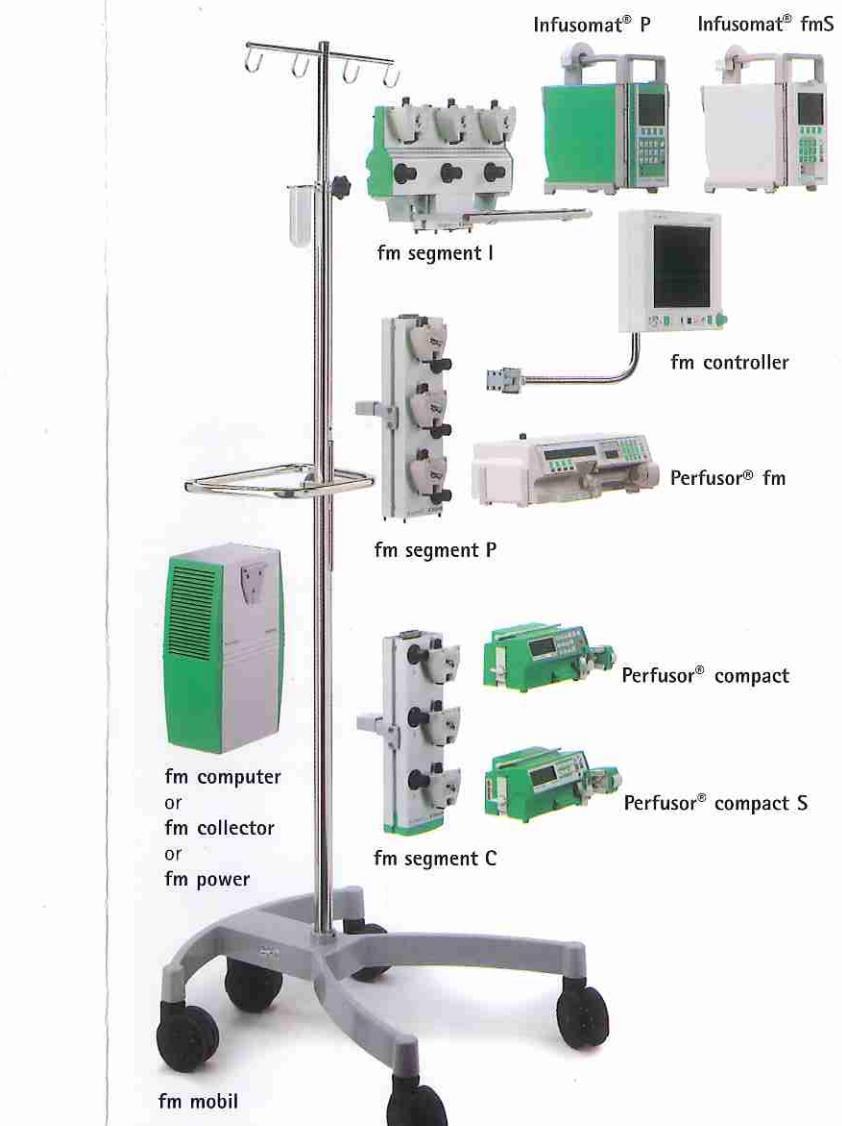

... customized to individual needs

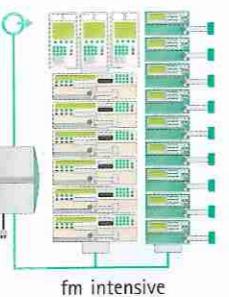Orderliness at the workplace, central power supply...

Workstation for up to 18 infusion pumps with the fm power - energy module.


Green circle icon with a black arrow pointing to the connection point.

... Data transfer to patient data management systems ...

Workstation for up to 9 infusion pumps with the fm collector - data module.


Green circle icon with a black arrow pointing to the connection point.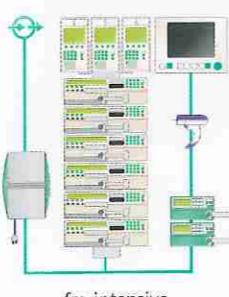

... Networked data communications, central management...

Network-compatible workstation for up to 9 infusion pumps with fm computer - CPU module, fm controller - control monitor, connection ports for 2 additional devices and bar code scanner.


Green circle icon with a black arrow pointing to the connection point.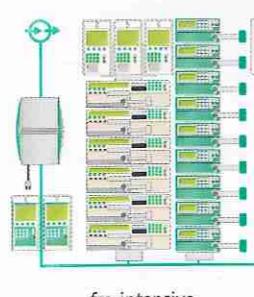

... for up to 18 infusion pumps...

Network compatible workstation for up to 18 infusion pumps with 2 fm computer - CPU modules (networked), fm controller - control monitor, connection ports for 4 additional devices and bar code scanner.

Green circle icon with a black arrow pointing to the connection point.

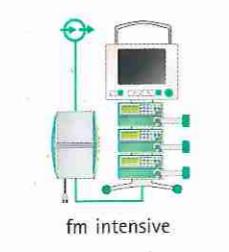... and for applications where mobility is the key

Mobile workstation, network-compatible, for 3 infusion pumps with fm computer - CPU module and fm controller - control monitor.

Green circle icon with a black arrow pointing to the connection point.

Design, adapt and expand a flexible and economical workplace

Workstations for the OR and ICU

One system fulfils all requirements: The modular system offers a diversity of configuration options for customizing the workstation.

More free space and orderliness

These workstations provide more room and mobility thanks to the economical arrangement of the infusion pumps: Perfusor® and Infusomat® are integrated into one ergonomic unit, power supply lines and data cables are reduced to a minimum, infusion lines are kept in order by guide elements.

The workstation can be easily adapted to equipment on the ward

Adapters make it easy to mount the workstations with ceiling suspension units or wall railing systems.

Maximum flexibility

It doesn't take much to adapt the workstations to the tasks at hand and the ambient conditions: Light-weight aluminium profiles with sturdy socket connections allow for multiple combinations. Three different modules for power supply and data communication offer step-by-step installation.

fm anaesthesia: Mobile workstation with VisiCon® patient monitor.

fm intensive: Workstation with 18 infusion pumps.

Flexible pump handling – with a selection of 5 different types

Five different system-compatible infusion pumps make it easy to configure the system as needed. The infusion pumps can be connected and removed at any time simply, quickly and safely. The system identifies them automatically, registers their data and logs them off when they are disconnected.

Original Infusomat® and Perfusor®

Infusomat® fmS – The volumetric infusion pump for maximum consistency in delivery.

Infusomat® P – Compatible with the standard PVC infusion sets in the Intrafix® P series.

Perfusor® fm – The precision syringe pump with automatic syringe lock-in.

Perfusor® compact – The handy syringe pump for every ward.

Perfusor® compact S – With very low delivery rates for paediatrics and neonatology.

DIANET® STAR Software:
For quick and reliable data communication.

fm anaesthesia –
Ranking Design Award.

Collect, transfer and process data quickly and reliably

Easy data entry by bar-coding

Time-saving and reliable: collect data on drugs and infusion rate without using a keyboard. By bar code-scanning the drugs are allocated to the infusion pumps and delivery rates suggested.

PC software supplies bar codes ...

A "Bar Code Editor" generates the bar codes. Drug names and delivery rates can be designated on an individual basis or taken from a drug list.

... and drug lists provided as a database


A "Drug List Editor" helps to edit the list of medicaments for a specific ward.

Optimal conditions for patient data management


All infusion-related data are recorded automatically, configured and reliably transferred to a Patient Data Management System (PDMS). Even when the pump configurations change, records are constantly updated.

Virtual systems make networking easy



Two programmes allow pre-testing and optimal preparation for system integration. The Windows-compatible programmes "BCCsim" and "BCCshow" offer simulation of the Patient Data Management System and fm workstations.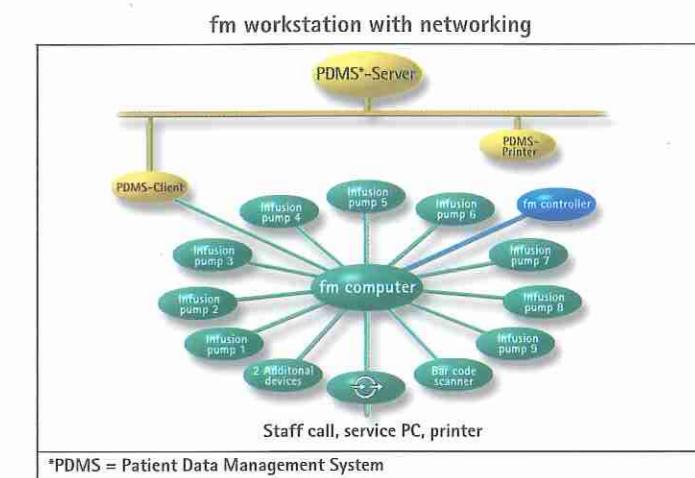

System integration: The time and effort expended within the PDMS are drastically reduced by pre-processing the data in the fm computer.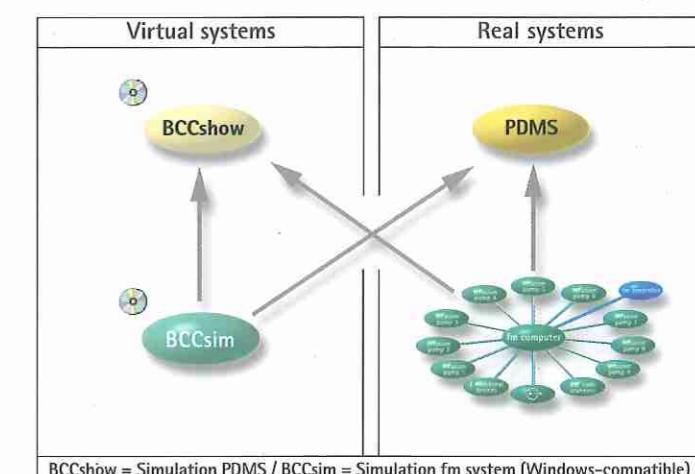

Simulation software: Drivers can be programmed quickly using data simulation.

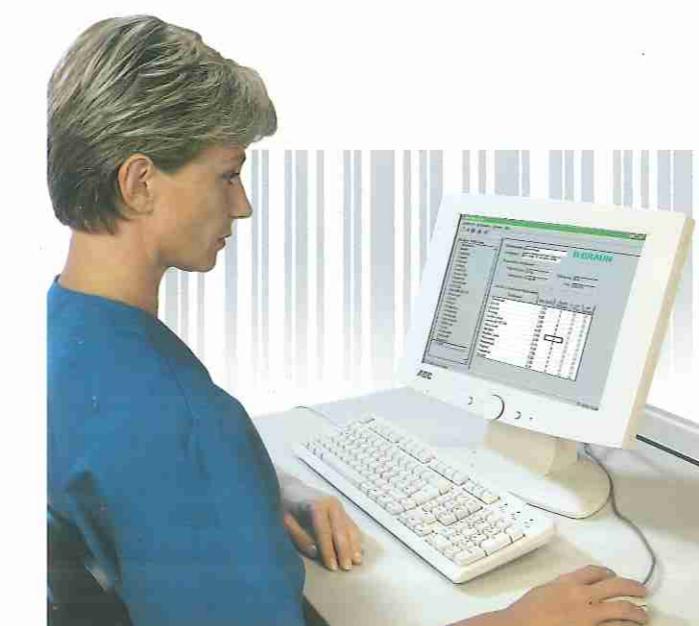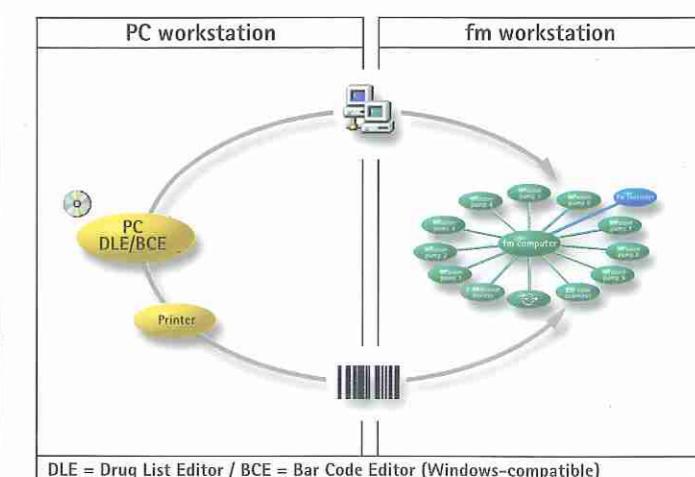Drug List Editor (DLE) and Bar Code Editor (BCE): The generation and management of drug-related information is easier thanks to task-oriented PC software.

Drug lists/Bar-coding: Drug lists specific to the ward can be loaded and allocated using the bar code scanner.

fm controller

B/BRAUN

R. Another
50 Years, ♂, 70 kg

4.10.2000
014-153-14

| | | | | | | | |
|------------------------------|-----------------------|------------------|-------------------------|--------------------------|----------------------|----------------------|----------------------|
| Aminophenyl HepatoID® | 41,6 ml/h | Lipofundin MCT20 | 20,8 ml/h | Glucose 10% | 10,4 ml/h | Propofol 10 mg/ml | 10,5 ml/h |
| | 500 ml -0.5h +0.5h | | 1.000 ml -0.5h +0.5h | | 50 ml -0.5h +0.5h | 50 ml -0.5h +0.5h | 50 ml -0.5h +0.5h |
| Dopamine 5 mg/ml | 1,8 ml/h | | | Midazolam | 0,0 ml/h | | |
| Epinephrine 0,2 mg/ml | | | | Mivacurium 2 mg/ml | 14,4 ml/h | | |
| Dobutamine 5 mg/ml | 2,1 ml/h | | | Clonidine 0,03 mg/ml | 1,56 ml/h | | |
| Dexmedetomidine 0,2 mg/ml | 1,8 ml/h | | | Clonazepam 0,01 mg/ml | 0,0 ml/h | | |
| Fentanyl 0,05 mg/ml | 0,0 ml/h | | | Insulin 1 IE/ml | 1,4 ml/h | | |
| Remifentanil 0,1 mg/ml | 2,0 ml/h | | | KCl 7,45% 1 mmol/ml | 2,0 ml/h | | |
| | -0.5h +0.5h | | | Furosemide 10 mg/ml | 1,0 ml/h | | |
| | | | | Heparin 300 IE/ml | 2,1 ml/h | | |

Control, monitor and document applications – simply and conveniently

fm intensive – central management of up to 18 infusion pumps ...

The fm controller sets new standards for operating convenience and safety. A single selector knob allows centralized coordination of the entire performance spectrum. All target functions are activated and operated single-handedly. The ultra high-contrast colour display guides the user through the straightforward menu structure and provides comprehensive information about patient status and trends.

... Get the clear picture of fluid balance


Major time savings thanks to automated documentation: All infusion-related data on the pumps are compiled automatically. No more complicated analyses of delivery rates. Additional fluid balance parameters can

be entered manually.
... Easy-to-adjust effective dose


No more complicated calculations, data input errors are minimised: Thanks to drug lists, all the user has to enter are the patient-specific settings. A proposed dose regimen is called up in just seconds and can be selected for administration.

fm anaesthesia – optimised propofol dosage


Achieve greater efficiency for intravenous anaesthesia: The desired propofol plasma levels are entered as target values using the function "OTCI" (Optimised Target Controlled Infusion). A pharmacokinetic algorithm automatically controls the delivery rates required to achieve and maintain these levels. Simply turn the selector knob to adjust the target value to the current status of the patient.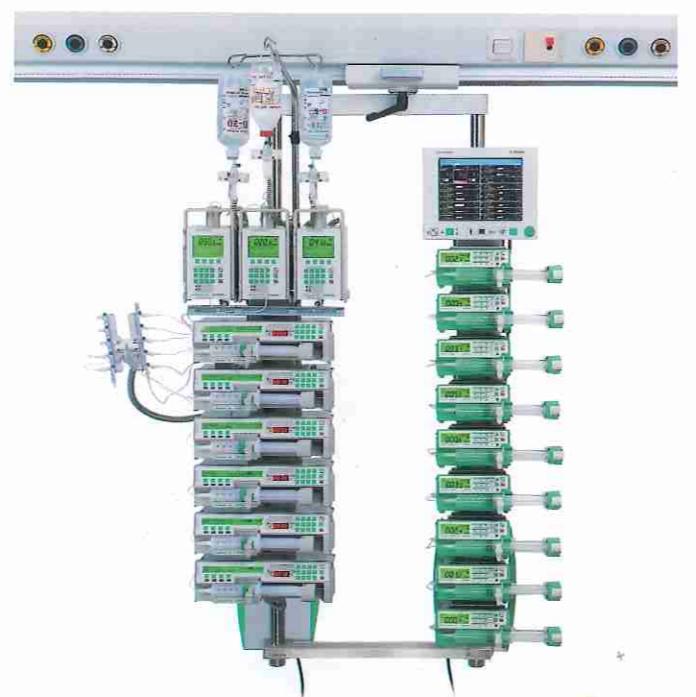


STATUS: Pump configuration, key parameters, short-term trends, alarms and Patient data.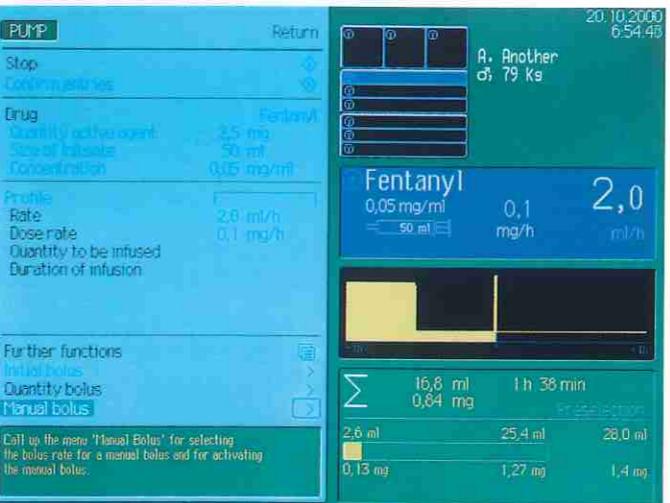


PUMP: Delivery parameters and Performance spectrum for settings. Typical profiles are available for setting automated infusion sequences.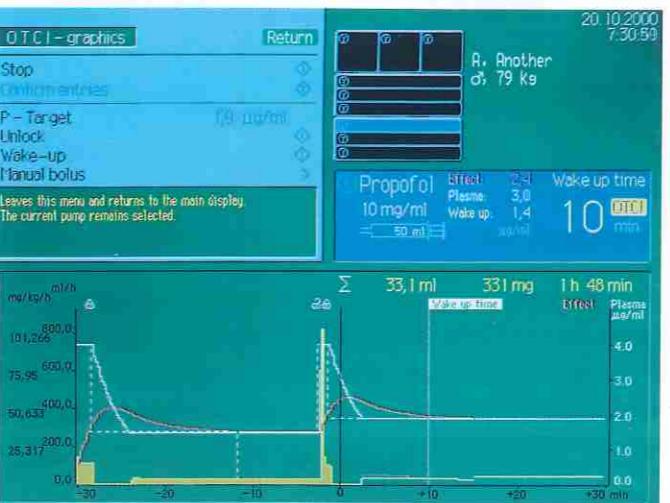


OTCI: Monitor drug level curves in the effective compartment and in the plasma including prognosis.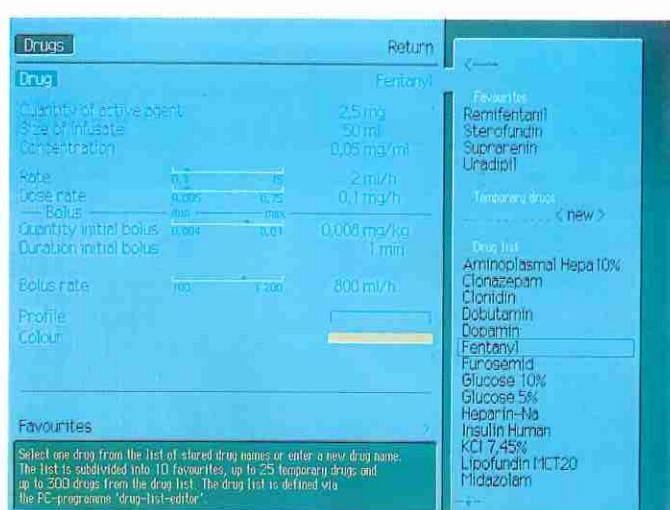

DRUGS: All characteristic data relevant to the infusion. Up to 300 drugs can be called up from the drug list.

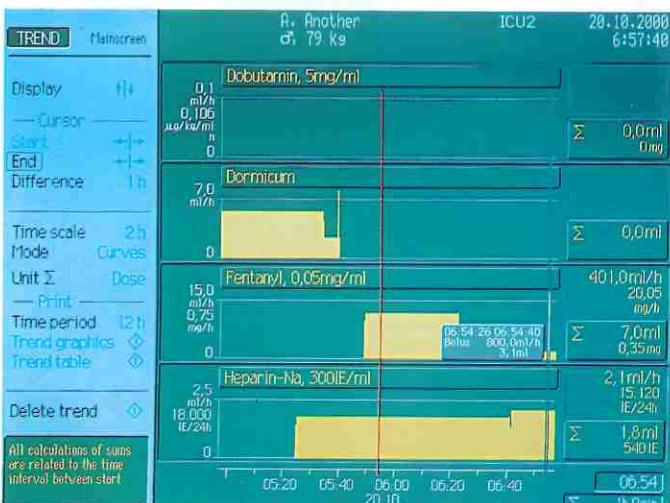
TREND: Graphs or tables for monitoring the delivery trends of up to 50 different drugs, including documentation of clinical course.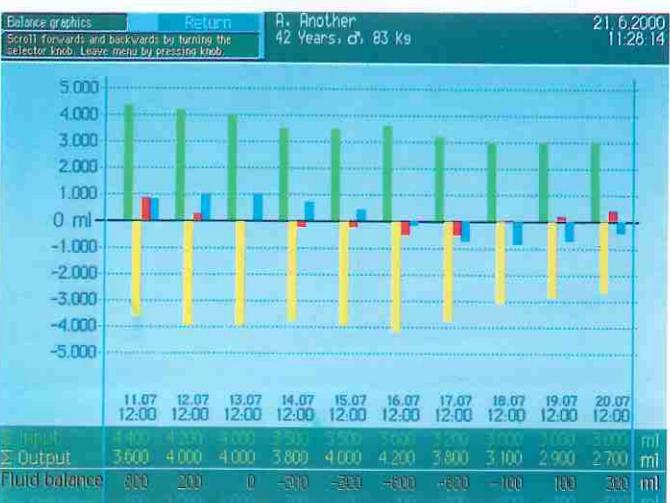

FLUID BALANCE: Complete fluid balance displayed in graphs or tables. Selectable periods of up to 31 days.

Manage the entire spectrum with the turn of one knob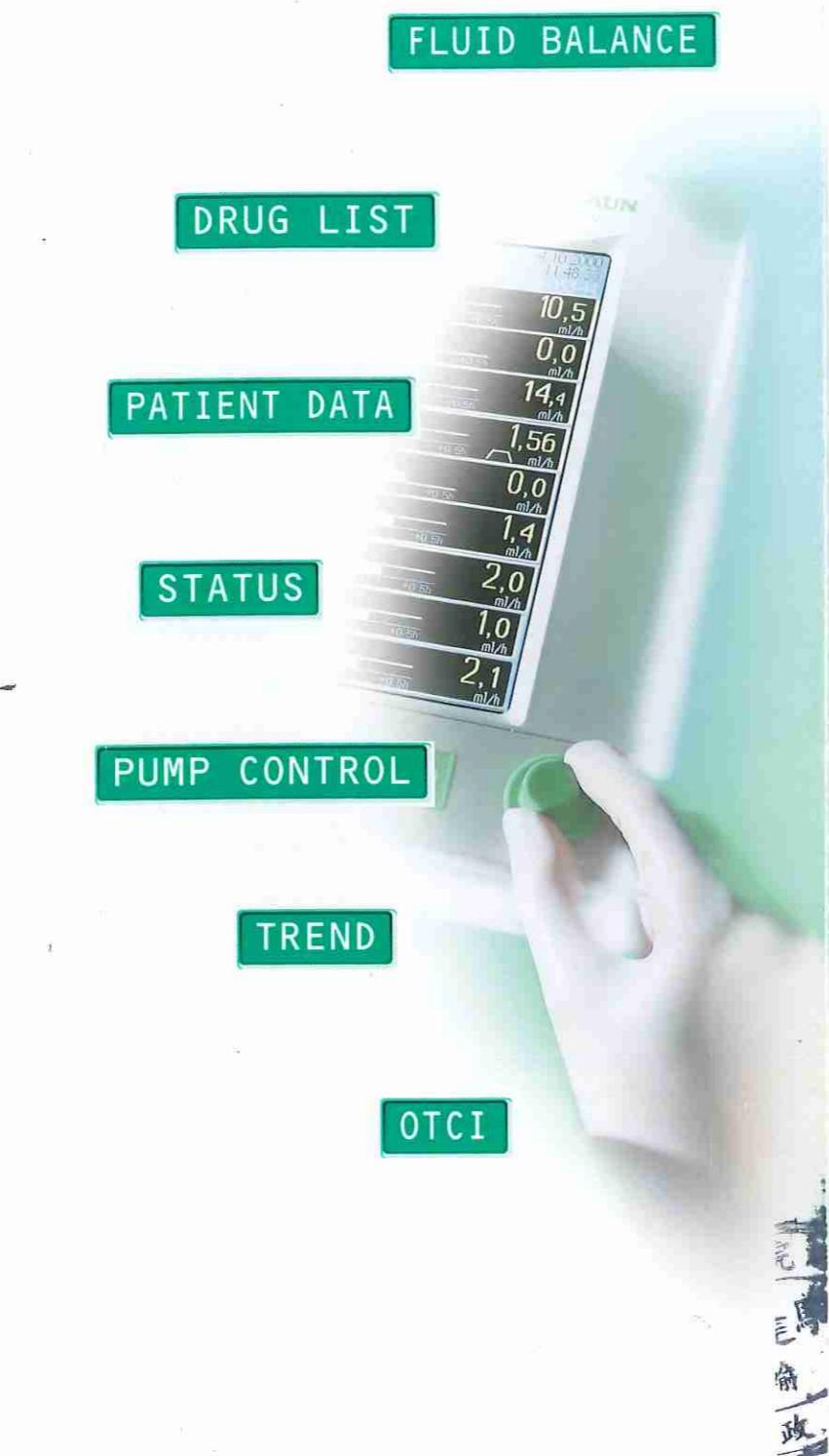

We'll help configure your fm workstation

| Infusion pumps | Order No./REF | Data cables | Order No./REF |
|---|------------------|---|---------------|
| Volumetric infusion pumps | | fm computer – fm controller | |
| Infusomat® fmS | 230V 8715548 | fm connect controller 0,5 | 8721220 |
| Infusomat® fmS | 200/240V 8715440 | 1,5 | 8721238 |
| Infusomat® fmS | 100/120V 8715416 | 3,0 | 8721246 |
| Infusomat® P | 230V 8712379 | fm segment – fm computer / fm collector / fm power | |
| Infusomat® P | 200/240V 8712212 | fm connect segment 0,5 | 8721254 |
| Infusomat® P | 100/120V 8712387 | 1,5 | 8721262 |
| Syringe pumps | | 3,0 | 8721270 |
| Perfusor® fm | 230V 8713901 | fm connect segment Y | 8721327 |
| Perfusor® fm | 200/240V 8713898 | fm computer | |
| Perfusor® fm | 100/120V 8713812 | fm connect computer 1,0 | 8721289 |
| Perfusor® compact | 230V 8714827 | 3,0 | 8721297 |
| Perfusor® compact | 110V 8714835 | fm connect pump 3,0 | 8721300 |
| Perfusor® compact S | 110/120V 8714886 | MFC connecting lead for staff call | 8711682 |
| Perfusor® compact S | 220/240V 8714843 | Adapters and mounting brackets | Order No./REF |
| Mounting profile for up to 3 infusion pumps | Order No./REF | fm adapt W, for mounting on wall railing systems, 2 per package | 8721211 |
| fm segment I | 8721050 | fm adapt N, for mounting on wall railing systems, 4 per package | 8721122 |
| For up to 3 Infusomat® fmS/Infusomat® P | | fm adapt I, for mounting a single fm segment I to round profiles Ø22 to 40 mm | 8721165 |
| fm segment P | 8721068 | fm clip, for connecting the fm computer or fm collector to mounting profiles | 8721203 |
| For up to 3 Perfusor® fm | | fm adapt R, for mounting to round profiles Ø22 to 40 mm, 2 per package | 8721149 |
| fm segment C | 8721076 | Trolley unit, feet | Order No./REF |
| For up to 3 Perfusor® compact/Perfusor® compact S | | fm stand, feet for fm segment | 8721157 |
| Modules for power supply/data communications | Order No./REF | fm mobil, trolley unit for up to 9 infusion pumps | 8721106 |
| fm computer, CPU module with data concentrator | 8721025 | Accessories | Order No./REF |
| Power supply for up to 9 Infusion pumps (3 segments) and fm controller | | fm support, infusion solution bag suspension | On request |
| - RS 232/RS 485 interface for PDMS, Ethernet optional | | fm order, tubing guide-way – Infusomat/Perfusor tubing | 8721173 |
| - Interfaces for bar code scanner, printer and servicing | | fm net, Ethernet option for fm computer | On request |
| - Connection ports for 2 additional devices | | fm energy, rechargeable battery pack for fm computer | On request |
| - Connection port for fm controller | | | |
| - Staff call | | | |
| - Connection port for additional fm computers | | | |
| fm collector, data module | 8721017 | | |
| Power supply for up to 9 infusion pumps (3 segments) | | | |
| - RS 232 interface for PDMS | | | |
| - Staff call | | | |
| fm power, energy module | 8721033 | | |
| Power supply for up to 18 infusion pumps (6 segments) | | | |
| - Staff call | | | |
| Control monitor, accessories | Order No./REF | | |
| fm controller, control monitor | 8721009 | | |
| High-contrast, high-resolution flat screen, Drug list, fluid balance, trend graphics, Central management of all pumps (requires DIANET®STAR software) | | | |
| fm segment S | 8721092 | | |
| Mounting profile with grip for fm controller | | | |
| fm hold, holding arm for fm controller | 8721130 | | |

B|BRAUN

HOSPITAL CARE

Headquarters for Asia Pacific Region:
B. Braun Medical Industries Sdn Bhd
(Company No. 19851-M)
Jalan Lepas 10, Industrial Zone
P.O. Box 800
10810 Penang, Malaysia
Tel.: 604-6122100
Fax: 604-6433750
www.braunap.com