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**BIONECT<sup>®</sup>**  
Hyaluronic acid sodium salt 0.2% Gel

30g tube

100g of gel contains hyaluronic acid sodium salt 200mg  
Provides an open and hydrate environment which rapidly permeate the skin layer suitable for wound healing, excellent in tissue regeneration after damage especially suited for post-surgery recovery of aesthetic surgery, effectively reduce wound healing time and prevent scar.  
Daily apply Bionect<sup>®</sup> Gel retains moisture, elasticity of skin and reduce wrinkles. Suitable for all skin types. For external use only.

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## BIONECT®

Hyaluronic acid sodium salt 0.2% Gel

Composition:  
Per 100g of gel contains  
hyaluronic acid sodium salt 200mg

Excipients:  
Sorbitol liquid, non-cristalisable,  
methyl p-hydroxybenzoate, propyl  
p-hydroxybenzoate, sodium dehyd-  
roacetate, carbomer 980, sodium hy-  
droxide, purified water.

Fidia Farmaceutici S.p.A.  
Via Ponte della Fabbrica  
3/A-35031 Abano Terme  
(PD) Italy

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## BIONECT®

0.2% Gel 30g

Active ingredient:  
Hyaluronic acid sodium salt 2mg/g  
(60mg per tube of 30g)

Description:  
Bionect® can

- easily penetrate stratum corneum to capture water in and keep skin moisturizing.
- Increase thickness of corium (ECM protein formed)
- improve skin elastosis and also maintain its pH value.

To use:  
Use in the morning and evening.  
Apply small amount of Bionect® Gel to your face and neck after thoroughly clean and dry your skin.

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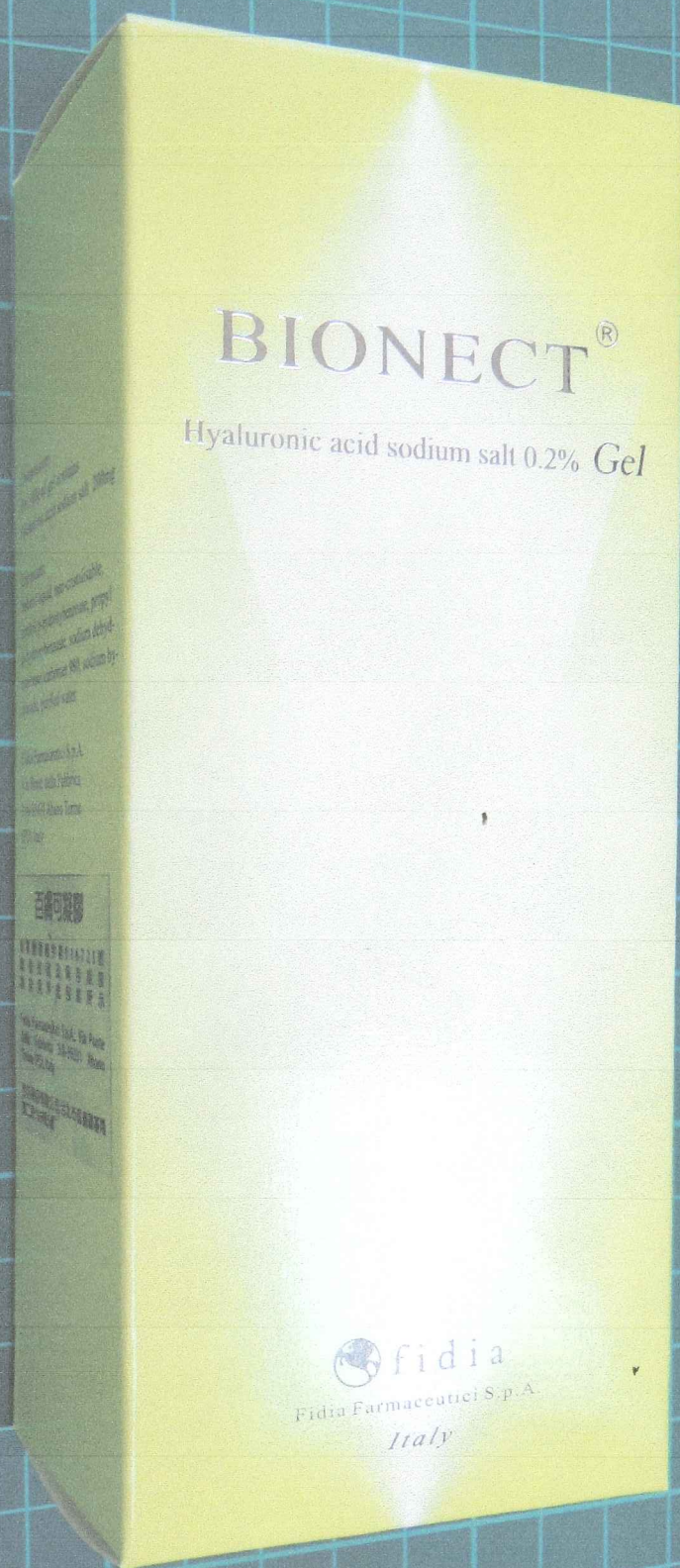
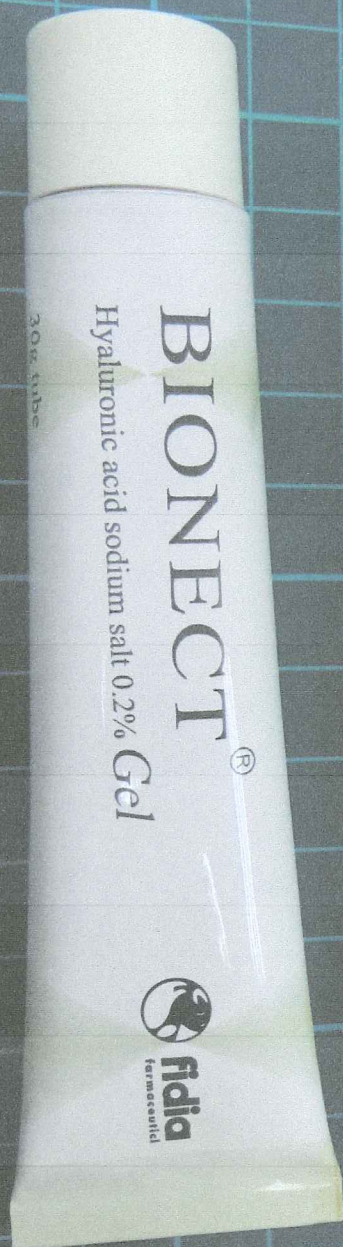
# BIONECT® Gel

## Hyaluronic acid sodium salt 0.2% gel

Bionect® Gel is hyaluronic acid sodium salt 0.2% gel. While infiltrating into the basal layer and corium of skin, it will enhance the proliferation and regeneration of the cell. That provides an excellent repair ability to overcome the internal and external cell damage factors.

With its incomparable small molecule, Bionect® Gel can easily and rapidly permeate the skin layer to increase corium thickness and decrease epidermis thickness. In addition, with this unique formula, it is able not only to retain moisture and elasticity of skin but also to nourish and revitalize dull skin in the purpose of reducing the appearance of fine lines.

Bionect® Gel is suitable for all skin types.

Component:  
Per 100g of gel contains  
hyaluronic acid sodium salt 200mg

Excipient:  
Solvent liquid, non-compressed  
methyl-p-hydroxybenzoate, propyl  
p-hydroxybenzoate, sodium lauryl  
sulfate, carbomer 980, sodium hyaluronate,  
distilled, purified water.

Fidia Farmaceutici S.p.A.  
Via Ponte della Fabbrica  
3/A-35051 Abano Terme  
(PD) Italy

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百得可凝胶  
Fidia Farmaceutici S.p.A. Via Ponte della Fabbrica 3/A-35051 Abano Terme (PD) Italy  
Fidia Farmaceutici S.p.A. Via Ponte della Fabbrica 3/A-35051 Abano Terme (PD) Italy

Exp. date

Lot n.

6104332

100g of gel contains hyaluronic acid sodium salt 200mg  
Provides an open and hydrate environment which rapidly permeate the skin layer suitable for wound healing, excellent in tissue regeneration after damage especially suited for post-surgery recovery of aesthetic surgery, effectively reduce wound healing time and prevent scar.  
Daily apply Bionect® Gel retains moisture, elasticity of skin and reduce wrinkles. Suitable for all skin types. For external use only.

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## Bionect® 0.2% Gel 30 g

### Bionect® 0.2% Gel

**Active ingredient:** Hyaluronic acid sodium salt 2mg/g (60mg per tube of 30g)

**Excipients:** sorbitol solution 70%, methyl p-hydroxybenzoate, propyl p-hydroxy-benzoate, sodium dehydroacetate, carbomer 980, sodium hydroxide, purified water.

### Presentation

Gel: tube of 30g

### What it is

Bionect® is a wound-healing agent used to treat wounds and cutaneous ulcers.

### Marketing Authorisation Holder

Med. Pharma. CO., LTD

### Manufacturer and final Controller

Fidia Farmaceutici S.p.A. - Via Ponte della Fabbrica 3/A - 35031 Abano Terme (PD) - Italy

### Why it is used

Bionect® stimulates the repair and regeneration of the skin, aiding the healing of abrasions, excoriations, superficial wounds, scalds, mild burns.

Bionect® is also indicated in localized outbreaks of sore skin caused by physical agents such as the sun, cold, wind and nappy rash, and in irritated and dry skin caused by radiotherapy treatments.

Bionect® can also be used as an aid in treatment for cutaneous ulcers of vascular origin, and slow-healing wounds such as bedsores.

### When it must not be used

Hypersensitivity to its components or other strictly related substances from a chemical point of view.

### What to do during pregnancy and breastfeeding

Bionect® can be used by pregnant and nursing mothers.

### Precautions

Cleanse and disinfect the area to be treated before applying Bionect®.

### What medical products or foodstuffs may modify the effect of Bionect®

No phenomena of interaction between Bionect® and antibiotics or other local treatments have ever been reported.

If you are using other medical products, ask the advice of your doctor.

### It is important to know that

Use of the product, especially if prolonged, may give rise to sensitization phenomena. Should this occur, consult your doctor before continuing the treatment.

The product does not interfere with the user's ability to drive or use machinery.

### How to use the product

Cleanse and disinfect the wound carefully before applying Bionect®. Treat the wound 2-3 times a day. Applying enough product to cover the entire area evenly.

### What to do if you have used too much

No manifestations of intoxication due to overdose of Bionect® have ever been reported.

The product is for external use only. In the case of accidental ingestion of an overdose of Bionect®, consult your doctor immediately or contact your nearest hospital.

### Undesirable effects

No systemic-type undesirable effects associated with Bionect® have even been reported.

Occasional reactions of a local type have been recorded, but they were always mild and resolved spontaneously. Use of the product, especially if prolonged, may give rise to sensitization phenomena. Following the instructions given in the package insert will reduce the risk of any undesirable effects. Should any such effects occur, however, it is advisable to consult your doctor.

It is important to inform your doctor of any undesirable effects that are not described in the package insert.

### Expiry and storage

The expiration date indicated on the packaging refers to unopened product that has been correctly stored.

**WARNING: do not exceed the recommended doses without consulting a doctor.**

**Do not use the product after the expiration date indicated on the packaging.**

**Keep out of the reach of children.**